SoftSurgicalStrip Chirurgische Tupfer (DE)

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SoftSurgicalStrip Surgical Sponges (EN)

ATTENTION!
The product is to be used by qualified users only!
Use only with undamaged packaging!

Product Description

SoftSurgicalStrip is made of a soft, pliable, moisture-absorbing material that is used to absorb fluid, protect tissue, and reduce bleeding.
SoftSurgicalStrip is equipped with a safety thread to facilitate retrieval.
SoftSurgicalStrip is optionally available with X-ray contrast thread to allow localization of the swabs.

Purpose / Indication for use

Indications for use of surgical swabs include removal of fluid accumulation in surgical wounds, control of bleeding, protection of tissue from mechanical tissue damage, and tissue moistening during prolonged surgical procedures in neurosurgery.

Contraindications

SoftSurgicalStrip is not intended to remain in the body after closure of the wound.
No other contraindications are known.

Recommended technique / application

Removal: Once the package is opened, the swabs may become contaminated. Therefore, remove the swabs from the package only immediately before use and dispose of opened packages after the operation. Immediately after opening the package, check the completeness of the swabs according to the number indicated on the package.
Placement: After opening the surgical area, open the appropriate size SoftSurgicalStrip pack and moisten the swabs using sterile water or physiological solution. Swabs then placed according to standard techniques and requirements. Apply sterile water or physiologic solution to swabs during the procedure to improve conformity to tissue contours and provide additional moisture to the tissue.
Count card: After removing the swabs from the surgical area, collect them by hooking the safety thread in the side notches on the count card. Re-check for completeness before closing the surgical area to ensure that all used swabs have been removed after surgical procedures.
Safety thread: The safety thread on the swabs is used only to locate the swabs and must not be used to pull them out.
X-ray contrast strip (optional): The X-ray contrast thread enables the swab to be localized in radiological imaging.

Warnings and safety instructions:

- The product is to be used by qualified personnel only.
- Product for ONE TIME USE - DO NOT reuse, repackage or re-sterilize the swabs.
- DO NOT use swabs from damaged packaging.
- CHECK PACKAGING: Not sterile if damaged, wet, seal broken, or past expiration date.
- DO NOT leave the swab in the body.
- DO NOT apply swab dry to wound area.
- DO NOT allow swabs to dry out during application.
- DO NOT remove swabs from the package until immediately prior to use and discard opened packages after surgery to avoid contamination.
- Use the counting card or use alternative counting methods to ensure that all used swabs have been removed after surgical procedures.
- DO NOT use the safety thread on the swab to pull it out.

Packaging / single use

SoftSurgicalStrip is supplied sterile unless this package has been damaged or opened.
Each package contains 10 pieces.
SoftSurgicalStrip sponges are to be used only once.

Shelf-life

The swabs must not be used after the expiry date (hourglass symbol) indicated on the packaging. Opened packages must no longer be used. Store in the original packaging.

The swabs have been radiation sterilized in the final packaging.

Storage Instructions

CelluStrip must be stored in an environment that maintains packaging and purity.
- Protect from moisture
- Protect from light
- Store in a temperature range from 4°C to 50°C

SoftSurgicalStrip is manufactured by:

neuro-competence GmbH
Birkenhügel 5
D-02689 Sohland a.d. Spree, Germany
Tel +49 (0) 35936-33 99-0
Fax +49 (0) 35936-33 99-11
E-mail sohland@neuro-competence.de

Should a serious incident or undesirable side effects occur during the handling and use of the product that are not listed in these instructions for use, please report them to us with a detailed description of the incident or side effects.
Please use the contact details above for this purpose.

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